



**TRADE NAME:** Endomax kit

**DESCRIPTION:** Filling material for root canals on a zinc-eugenol basis.

**INDICATIONS FOR USE:** Filling of root canals of all groups of teeth (using gutta-percha posts).

**AREA OF USE:** Dentistry.

**POTENTIAL CONSUMERS:** The qualified medical personnel.

**CONTRAINDICATIONS:** Hypersensitivity to one of the components. Risk of allergy to any of the components, especially in case of contact with the gums. Do not use inappropriately.

**CHARACTERISTICS:** In order for the pastes to act as ideal materials for root canal fillings, they must not resorb after hardening and be drawn out over many years. These pastes must be radiopaque and easy to insert into the root canal. Endomax+ powder has all of the above qualities. In addition, the product has adjuvant anti-inflammatory activity due to the presence of hydrocortisone acetate.

**INGREDIENTS: (Powder 15g+Liquid 10ml)**

Powder (Hydrocortisone acetate, Filler:Diiodothymol, Barium sulphate, Zinc oxide, Magnesium stearate)

LIQUID (Eugenol, Peppermint oil, Anise oil)

**PACKAGE CONTENTS**

- 1 piece x 15 g of powder
- 1 piece x 10 ml of liquid
- 1 piece x dropping tube
- 1 piece x scoop
- 1 piece x mixing pad
- 1 pieces x instruction for use

**PRECAUTIONS:** Avoid prolonged contact with skin, eyes and soft tissues. Rinse thoroughly with water after contact. Do not use with methyl methacrylate-based dental materials. Do not use for luting composite restorations.

**GESTION:** The tooth must be well insulated (with cotton rolls or cofferdam) After thoroughly treating the root canal, disinfecting it with an antiseptic solution, followed by rinsing and drying with paper pins, prepare the paste.

Shake the powder before kneading to ensure an evenly distributed homogeneous mass. Prepare the paste on a notepad or glass plate. Mix the powder and liquid thoroughly until a soft paste (consistency of sour cream) is obtained.










**WARNING:** For good results we recommend mixing 2 parts powder (measured by the measuring spoon) with 3-6 drops liquid (it can be eugenol or Endomax kit+ liquid)





Proceed to fill the root canal with the canal filler and introduce one or more pre-prepared pins into the canal. Ready paste is introduced in two or three portions into the prepared root canal.

**TERMS OF STORAGE:** Store in a dry place protected from light at +4 to +24° C.  
 Period of validity 3 years from the date of production

**TERMS OF TRANSPORTATION:** Conditions of transport All types of covered, dry, clean and odourless vehicles and the transport regulations applicable to the mode of transport.

**Applicable Symbols**

	<p>Manufacturer</p>
	<p>Date of production</p>
	<p>Protect from moisture</p>
	<p>Avoid exposure to sunlight</p>
	<p>Upper limit of temperature range</p>
	<p>Refer to instruction for use</p>
	<p>Non- sterile product</p>
	<p>Caution! Refer to instruction for use</p>
	<p>European conformity mark to standards and identification number of authorized Authority. The product conforms to the essential requirements of the Medical Devices Directive (93/42/EEC)</p>

	Lot number
	Medical Device
	Do not use if the package is damaged
	Shelf life M- month Y- year

**ENVIRONMENTAL PROTECTION REQUIREMENTS:** Products at room temperature do not emit toxic substances into the environment and do not have a harmful effect on the human body upon direct contact.

**STERILITY INFORMATION:** Non-sterile

**MAINTENANCE AND REPAIR:** Nonrepairable

**WARRANTY:** The manufacturer guarantees the quality of the medical product until its expiry date, provided the conditions for use, transport and storage are met.

**UTILIZATION:** The product does not belong to hazardous waste. To utilize according to the standard medical practice and applicable local, state and federal laws and regulations. A hazard class for medical waste taking into account the specification of morphological structure: Class A

**MEDICAL PRODUCT MANUFACTURER (DEVELOPER)**

Queen Dental GmbH  
 Neue Straße 67, 99846 Seebach, Germany  
 Neo Strasse 67, 99846 Seebach, Germany  
 Phone: +49 36929 133948  
 E-mail: info@queen-dental.de

**PLACE OF MANUFACTURE OF A MEDICAL PRODUCT**

Neue Straße 67, 99846 Seebach, Germany  
 Neo Strasse 67, 99846 Seebach, Germany