

**Trade name:** Endomax kit

**Product description:**

Cement for root canal filling

**Indications for use:**

Filling of root canals of all groups of teeth (using gutta-percha posts).

**Composition:**

Powder: hydrocortisone acetate, filler: diiodothymol, barium sulphate, zinc oxide, magnesium stearate.

Liquid: eugenol, peppermint oil, aniseed oil).

**Directions for use:**

The tooth must be well insulated (using cotton rolls or cofferdam) After thoroughly treating the root canal, disinfecting it with an antiseptic solution, followed by rinsing and drying with paper pins, prepare the paste.

Shake the powder before kneading to ensure an even distribution of a homogeneous mass.

Prepare the paste on a pad or glass plate. Mix the powder and the liquid thoroughly until a soft paste (consistency of sour cream) is obtained.

For good results it is recommended to mix 2 parts of the powder (measured with a measuring spoon) with 3-6 drops of the liquid.

Proceed to fill the canal with a canal filler and insert one or more pre-prepared pins into the canal. The ready paste is introduced in two or three portions into the prepared root canal.

**Package contains:**

1 x 15 g powder

1 x 10 ml liquid

1 x spoon

1 x mixing pad

1 x instructions for use

**Contraindications:**

Hypersensitivity to one of the components of the material. Risk of allergy to any of the ingredients, especially in the case of contact with the gums. Do not use inappropriately.

**Precautions:**

Avoid prolonged contact with skin, eyes and soft tissues. Rinse thoroughly with water after contact. Do not use with methyl methacrylate based dental materials. Do not use for luting composite restorations.

**Storage conditions:**

Store in a dry place protected from light at a temperature of +4 to +24° C.

**Shelf life :**

3 years from date of manufacture.

**Disposal:**

This product is not hazardous waste. Dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations. Hazard Class for medical waste with morphological specification: Class A

**Manufacturer:**

Queen Dental GmbH

Neue Straße 67, 99846 Seebach, Germany

Phone: +49 36929 139999

E-mail: sales@queendental.de