

Trade name: Magic Bond.

Description:

Light-curing universal self-etching adhesive.

Promotes maximum preservation of tooth tissues, allows to isolate the pulp from all types of irritants and achieve high adhesion rates of dental materials with teeth hard tissues.

Indications:

- Bonding of composite materials, compomers for direct dental restoration
- Bonding of inlays, onlays, veneers and crowns made of ceramics and composites for indirect restorations

Contraindications:

In rare cases, some patients have been observed to be hypersensitive to the material. In case of allergic reactions, it is necessary to stop using the material and consult a doctor.

Composition:

HEMA, demethacrylate resins, ethanol, water, 10-MDP phosphate monomer, phosphoric acid, copolymer, silane, initiators

Application:

Route of administration

Perform tooth preparation using standard techniques. Use a rubber dam to protect the adjacent teeth.

Dry the prepared surfaces by blowing them with air without impurities.

Note: Use calcium hydroxide to protect the pulp.

Placement Technique:

- Shake the bottle before use
- Apply a few drops of material on the clean mixing pad
- Close the bottle tightly with the lid
- Apply the material to the conditioned enamel and dentin surface using the applicator tip.
- Let the material dry for 5-10 seconds.
- Carefully dry a surface within 5 seconds

Note:

After long storage of material in the refrigerator, before using it, let the bottle stand for a few minutes at room temperature.

Interaction with other materials:

It is impossible to use together with the laying containing eugenol as phenol interferes with polymerization.

CAUTION: Warnings

Keep out of the reach of children!

For dental use only!

Wear suitable protective clothing and gloves before use.

Avoid contact with eyes. In case of contact with eyes, rinse with plenty of water.

Wear protective gloves.

If allergic reactions occur in particularly sensitive patients, the material should be removed and its further use should be discarded.

Completeness

-Package contents:

1 x bottle 6 g

1 x instruction for use

Storage conditions

Store at a temperature from 5 °C to 25 °C; relative humidity of 10-95%

Avoid exposure to sunlight.

Protect from moisture.

Close the syringe right after use to avoid light influence.

Period of validity:

Shelf life of material 3 years. Do not use after the expiry date.

Utilization

Utilization, according to MPI rules for class A non-hazardous medical waste.

Manufacturer of a medical device

Queen Dental GmbH ("Queen Dental GmbH")

Neue Straße 67, 99846 Seebach, Germany

Neo Strasse 67, 99846 Seebach, Germany

Phone number: +49 36929 139999

E-mail: sales@queen-dental.de