

Trade name: Queenetch

Description:

Queenetch is used for etching enamel and dentin, for better adhesion of restoration materials to the tooth in the process of tooth preparation for filling

Indications:

Etching of enamel before filling.

Contraindications:

Intolerance to one of the components of the material, hypersensitivity.

Composition:

Phosphoric acid 38%, double distilled water, silica gel, pigment.

Application:

Apply a small amount of gel with an applicator or using a dispensing tip on the prepared, washed and dried enamel surface (reaction time of 30 seconds) or dentin (reaction time of 15 seconds). For enamel and dentin, the total etching time is 30-45 seconds. Afterwards rinse the gel thoroughly with a water-air jet and dry gently.

The etched enamel surface should have a chalky dull appearance; If not, repeat the etching procedure. The etched surface should then be primed / adhesive and the restoration proceeded.

Re-cap the gel tightly after each use.

CAUTION: Warnings

Keep out of the reach of children!

For dental use only!

Wear suitable protective clothing and gloves before use.

Avoid contact with eyes. If contact with the eye occurs, rinse immediately with plenty of water.

To reduce the risk of an allergic reaction, shorten the contact time with these materials. In case of contact with skin, immediately wash with water and soap and rinse thoroughly.

It is recommended to wear protective gloves and non-contact equipment. Acrylates can penetrate normal medical gloves. If restorative material comes into contact with the glove, remove and throw it out, wash your hands immediately with soap and water and put on a new glove. If you have an allergic reaction, seek medical attention.

Package Contents:

- etching gel "Queenetch" in the syringe, 5 g – 1 piece;

- instruction for use - 1 piece.

Storage conditions

Store at a temperature from 5 °C to 25 °C; relative humidity of 10-95%

Avoid exposure to sunlight.

Protect from moisture.

Close the syringe right after use to avoid exposure to sunlight.

Period of validity:

Shelf life of material 3 years. Do not use after the expiry date.

Utilization

Utilization, according to MPI rules for class A non-hazardous medical waste.

Manufacturer of a medical device

Queen Dental GmbH ("Queen Dental GmbH")

Neue Straße 67, 99846 Seebach, Germany

Neo Strasse 67, 99846 Seebach, Germany

Phone number: +49 36929 139999

E-mail: sales@queen-dental.de