

Trade name: Queenetch

Indications:

Queenetch is used for etching enamel and dentin, for better adhesion of restoration materials to the tooth in the process of tooth preparation for filling

Contraindications: Intolerance to one of the components of the material, hypersensitivity. Do not misuse.

Can cause irritations as a result of direct contact with a pulp. To avoid irritations from direct contact with pulp the area around the pulp should be covered with a thin layer of calcium hydroxide.

Composition:

Queenetch Gel for tooth treatment: phosphoric acid 37%, double distilled water, silica gel, pigment.

Application:

Apply a small amount of gel with an applicator or using a dispensing tip on the prepared, washed and dried enamel surface (reaction time of 30 seconds) or dentin (reaction time of 15 seconds). For enamel and dentin, the total etching time is 30-45 seconds. Afterwards rinse the gel thoroughly with a water-air jet and dry gently.

The etched enamel surface should have a chalky dull appearance; If not, repeat the etching procedure. The etched surface should then be primed / adhesive and the restoration proceeded.

Re-cap the gel tightly after each use.

CAUTION: Warnings

Keep out of the reach of children!

For dental use only!

Wear suitable protective clothing and gloves before use.

Avoid contact with eyes. If contact with the eye occurs, rinse immediately with plenty of water.

To reduce the risk of an allergic reaction, shorten the contact time with these materials. In case of contact with skin, immediately wash with water and soap and rinse thoroughly.

It is recommended to wear protective gloves and non-contact equipment. Acrylates can penetrate normal medical gloves. If restorative material comes into contact with the glove, remove and throw it out, wash your hands immediately with soap and water and put on a new glove. If you have an allergic reaction, seek medical attention.


Use with caution in patients with mental disorders.


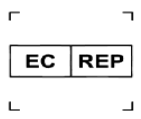









Do not use materials for pregnant women (due to insufficient clinical data)




Package Contents

- etching gel "Queenetch" in the syringe, 5 g – 1 piece;
- instruction for use - 1 piece.

Applicable Symbols

	<p>Manufacturer</p>
---	---------------------

	<p>Date of production</p>
	<p>Authorized representative in the European community</p>
	<p>Protect from moisture</p>
	<p>Avoid exposure to sunlight</p>
	<p>Upper limit of temperature range</p>
	<p>Refer to instruction for use</p>
	<p>Non- sterile product</p>
	<p>Caution! Refer to instruction for use</p>
	<p>European conformity mark to standards and identification number of authorized Authority. The product conforms to the essential requirements of the Medical Devices Directive (93/42/EEC)</p>
	<p>Lot number</p>
	<p>Medical Device</p>

	Do not use if the package is damaged
	Shelf life M- month Y- year
	Eurasian Conformity mark of medical products for circulation in the market of the Eurasian Economic Union

Transportation

All types of the covered transport at a temperature from 5 °C to 25 °C; relative humidity of 10-95%

Storage conditions

Store at a temperature from 5 °C to 25 °C; relative humidity of 10-95%

Avoid exposure to sunlight.

Protect from moisture.

Close the syringe right after use to avoid exposure to sunlight.

Period of validity:

Shelf life of material 3 years. Do not use after the expiry date.

Manufacturer's guarantees

Queen Dental GmbH ("Куин Дентал ГмбХ ") guarantees, that this product meets the requirements of all published specifications, under observance of storage, transport and use regulations. If you have any suspicions about the quality of this medical device, you should contact an authorized representative on the territory of the Russian Federation.

Utilization

Utilization, according to MPI rules for class A non-hazardous medical waste.

MANUFACTURER (DEVELOPER) OF A MEDICAL DEVICE

Queen Dental GmbH ("Queen Dental GmbH")

Neue Straße 67, 99846 Seebach, Germany

Neo Strasse 67, 99846 Seebach, Germany

Phone number: +49 36929 133948

E-mail: info@queen-dental.de

PLACE OF PRODUCTION OF THE MEDICAL PRODUCT

Neue Straße 67, 99846 Seebach, Germany

Neo Strasse 67, 99846 Zeebakh, Germany